

# 3Shape TRIOS®

Doctors with a TRIOS® scanner can send digital impressions directly to Vanguard through 3Shape Communicate.

Dentists should be registered with a 3Shape Communicate account when the scanner is installed. If Vanguard Prosthetics is not already pre-installed on the list of available labs, send a Connection Request from the Communicate account to Vanguard Prosthetics.

Note: Dentists can register themselves at <http://portal.3ShapeCommunicate.com>. Vanguard Prosthetics will respond that the lab is TRIOS ready to receive your scans.

- 1) After selecting the patient name, press New Session.
- 2) The lab list will appear, select Vanguard Prosthetics.
- 3) Complete the order form information. In the Comments, specify specific materials, and if the case is model-less.
- 4) Complete the scanning process and press the Send Order envelope icon.
- 5) On the Send Order screen, verify the case information and scans, and then press Send Order.



819 SE 14th Loop Suite 125, Battle Ground, WA 98604 | 360-342-8303 | [vanguardprosthetics.com](http://vanguardprosthetics.com)



3Shape TRIOS®

Should you have questions regarding the products, services and/or processes associated with submitting digital files to Vanguard Prosthetics, please contact us directly at 360.342.8303 or [support@vanguardprosthetics.com](mailto:support@vanguardprosthetics.com).